

IMPORTANT INFORMATION - PLEASE READ AND SAVE



Litter Cover

Made in USA
of US and foreign components

USE

The design of the CMC Litter Cover incorporates extra room to accommodate a litter with a CMC Litter Shield, medical bags, or other supplies. If desired, tightening the two compression straps while tucking the extra material inside can eliminate the extra room.

The vinyl coated material used to make the CMC Litter Cover will provide a high degree of protection from the sun and rain. **Please Note: This cover is not waterproof. Moisture can enter through the seams and zipper and can become trapped inside. Storing the litter with the zippers down will help to prevent this, however, equipment should be checked and dried as needed to prevent damage.**

CARRYING, MAINTENANCE AND STORAGE

During use, carrying and storage keep the equipment away from acids, alkalis, exhaust emissions, rust and strong chemicals. Do not expose the equipment to flame or high temperatures. Carry the equipment where it will be protected as the equipment could melt or burn and fail if exposed to flame or high temperatures.

If the equipment becomes soiled, it can be washed in cold water with a mild detergent that is safe for use with nylon and polyester. Dry out of direct sunlight. Do not dry in an automatic dryer. Store in a cool, dry location. Do not store where the equipment may be exposed to moist air, particularly where dissimilar metals are stored together.

REPAIR

All repair work shall be performed by the manufacturer. All other work or modifications void the warranty and releases CMC from all liability and responsibility as the manufacturer.

CMC Rescue, Inc.

6740 Cortona Drive, Goleta CA 93117
USA
805-562-9120 / 800-235-5741
cmcpro.com

ISO 9001 Certified

©2018 CMC Control No. 724104.00.012418